

Sticky Mitt




Pre-Op Glove

Order Ref: STMT200



To facilitate the removal of hair and lint from the patients skin

- Provides an efficient way of removing loose hairs and debris during pre-op surgery preparation
- Quick application with a large adhesive area
- Optimised adhesion reduces the risk of skin irritation
- Can be used on both curved and flat areas
- Can help to surgical site contamination risk
- Helps reduce the risk of post procedural contamination/infection
- Latex free
- Easy disposal after use

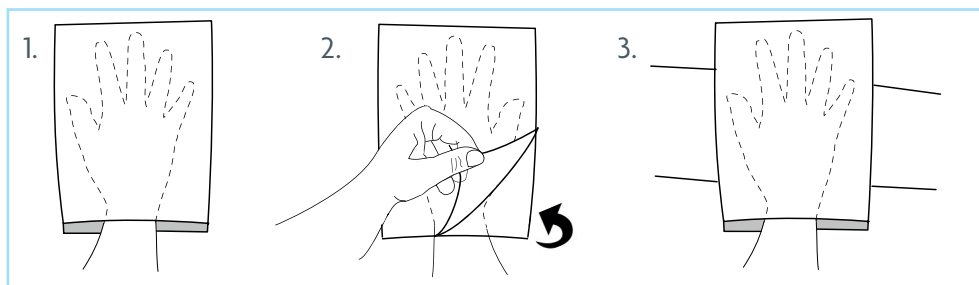

Instructions for use:

1. Insert hand into the open end, with the palm facing the backing paper.
2. Turn over and gently peel the backing paper
3. Remove the loose hairs or lint with the adhesive surface.

Material Code	Description	Box Qty
STMT200	Sticky Mitt Pre-Op Glove	200